

Regd. Office :


A- 6, NAND Complex,
Near AIMS Oxygen, Juna Padra Road,
Baroda 390 020 - INDIA

Marketing / Customer Support :

5-6, 3rd Floor, Mrudula Sadan,
Pratap Road, Raopura,
Baroda 390 001 - INDIA

JUTRON Vision

enhancing vision ~ enriching life

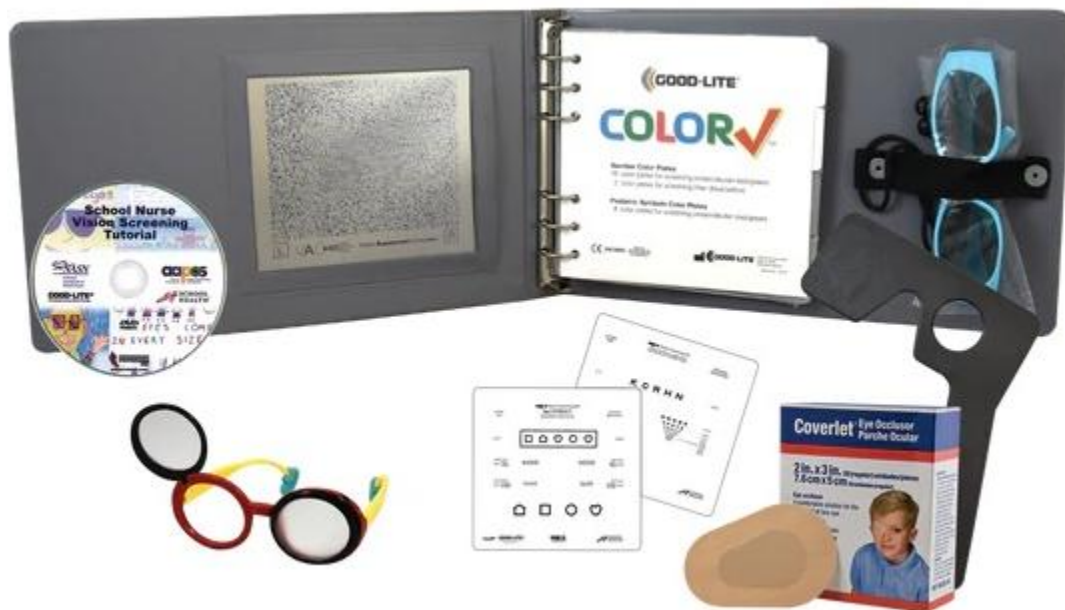

130 AAPOS Supplemental Screening Kit

The American Association for Pediatric Ophthalmology and Strabismus (AAPOS) is offering a supplemental basic screening kit to assist anyone who screens vision, including school nurses, Lions Club members, Head Start, and medical personnel working in pediatrics and family medicine.

The AAPOS Vision Screening Kit: Supplemental Screening Package is for What Ages?

The AAPOS Vision Screening Kit: Supplemental Screening Package is for screening the vision of individuals beginning at age 3 through adulthood.

Regd. Office :

A- 6, NAND Complex,
Near AIMS Oxygen, Juna Padra Road,
Baroda 390 020 - INDIA

Marketing / Customer Support :

5-6, 3rd Floor, Mrudula Sadan,
Pratap Road, Raopura,
Baroda 390 001 - INDIA

JUTRON Vision

enhancing vision ~ enriching life

What's in the Kit?

- Spiral-bound, multisided, near eye charts with attached 16-inch cord.
 - Sloan Letters for when children know their letters.
 - LEA Symbols for preschoolers and for children who do not yet know their letters.
- Full-Line Threshold and Single-Line Critical Line formats:
 - Threshold screening involves beginning at the top of an eye chart and moving down the chart until the child or adult misses 3 of 5 optotypes.
 - Critical line screening is using only the line which the child or adult should correctly identify 3 of 5 optotypes according to age.
 - Children ages 3 and 4 years should correctly identify 3 of 5 optotypes on the 20/40 line.
 - Children aged 5 years and older, and adults, should correctly identify 3 of 5 optotypes on the 20/32 line.
- Coverlet Adhesive Eye Occluder.
- Black hand-held flip paddle occluder for children aged 10 and older and adults who will not tolerate patches.
- Flip-Up Occluder Glasses for children under age 10 years who will not tolerate patches.
- Random Dot Butterfly stereopsis test with intermediate-sized polarized glasses.
- 16-Plate pseudoisochromatic color vision test.
- Instructions printed on eye chart card.