

Regd. Office :

A- 6, NAND Complex,
Near AIMS Oxygen, Juna Padra Road,
Baroda 390 020 - INDIA

Marketing / Customer Support :

5-6, 3rd Floor, Mrudula Sadan,
Pratap Road, Raopura,
Baroda 390 001 - INDIA

JUTRON Vision

enhancing vision ~ enriching life



Part # 346700 Bagolini Striated Lenses

- Finely striated, clear lenses used for evaluating retinal correspondence in patients with eye deviations.
- Consists of 2 circular lenses, indexed and mounted in 38mm trial lens metal rings with tabs for use by the examiner.
- Used in the analysis of anomalous correspondence, suppression etc.
- In testing suppression, the lenses are placed in a trial frame with the striations at an angle of 135° and 45° for each eye respectively.
- Then with a spotlight stimulus, such as a penlight or transilluminator, a patient with no suppression will see two diagonal lines crossing above, below or at the light source.
- This can be accomplished at near or far distance. If suppression is present, all or part of one line will not be seen.
- If the diagonal lines cross at the source when the cover test indicated an ocular deviation, the patient has harmonious, abnormal retinal correspondence. Sold as a pair.