

**Regd. Office :**

A- 6, NAND Complex,  
Near AIMS Oxygen, Juna Padra Road,  
Baroda 390 020 - INDIA

**Marketing / Customer Support :**

5-6, 3rd Floor, Mrudula Sadan,  
Pratap Road, Raopura,  
Baroda 390 001 - INDIA

**JUTRON Vision***enhancing vision ~ enriching life*

## **# 477700      Vergence Facility Prism 12D/3D – Mounted**

These vergence prisms quickly screen for binocular vision. Use these to get the patient to cycle between the two prisms up to 15 cycles per minute. Use at 16" (40 cm) with a 20/30 near point line. Prisms are 12D base-out and 3D base-in and 1.5" high (38mm).

The Vergence Facility Prism is available in :  
Mounted to a stainless steel holder