

Regd. Office :


A- 6, NAND Complex,
Near AIMS Oxygen, Juna Padra Road,
Baroda 390 020 - INDIA

Marketing / Customer Support :

5-6, 3rd Floor, Mrudula Sadan,
Pratap Road, Raopura,
Baroda 390 001 - INDIA

JUTRON Vision

enhancing vision ~ enriching life

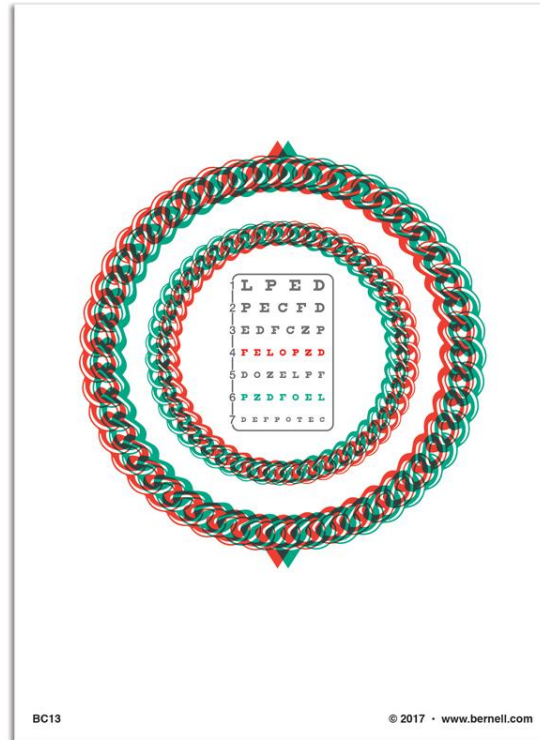

BC13 Bracelet Fixed Tranaglyph™

The Bracelet Tranaglyph is an excellent training device for fusional and suppression therapy.

The small bracelet circle should appear to stand forward and the larger behind the letters. If using the reversible spectacles and the red lens is over the left eye instead of the right, this position is reversed, and so is the eyes' relationship to lines 4 and 6. Line 4 is seen only by the left eye, and line 6 is seen only by the right eye, these are the suppression checks. The other lines are seen by both eyes.

4" x 5.5" red/green targets are on transparent vinyl. Works with 550 holder or dual polachrome orthopter.

Size: 4" x 5.5"

Material: Red/Green slide on transparent vinyl

Usage: Can be used with Dual Polachrome Trainer or other slide holders.

Chain Rings:

Outer Ring: 2mm of separation (Red to left and green to right) - Inward

Inner Ring: 2mm of separation (Green to left and red to right) - Outward

Center Eye Chart Acuity:

Line 1: 20/70

Line 2: 20/60

Line 3: 20/50

Line 4-6: 20/40

Line 7: 20/30